



## Specification

### General

The Powerex Scroll Enclosure Systems are designed to provide medical breathing air for hospital and medical institutions. This system meets NFPA 99 requirements for Risk Category 1 systems. Each system is completely tested before shipment and includes:

- Multiple oil-less scroll air compressors and associated equipment mounted inside a rigid steel sound enclosure (number of compressor modules vary depending on size of system).
- Master control panel.
- · Corrosion resistant air receiver.
- Redundant medical desiccant air dryers with purge control.
- Dew point and CO monitors.

System installation requirements:

- Main power connection to each individual compressor box.
- Interconnecting power wiring between master panel and tank/ dryer skid.
- Control signal wiring between master panel and each compressor unit.
- Remote inlet air connection to each module.
- Interconnecting plumbing connections between each compressor and tank/dryer skid.

The field connections required will be system intake, exhaust, power connection, and interconnecting piping and wiring. Vibration isolation pads are included with the system.

## **Oil-less Scroll Compressor Pump**

Each compressor shall be belt driven oil-less rotary scroll single stage, air-cooled construction with absolutely no oil needed for operation. Direct drive compressors shall not be used.

- The rotary design shall not require any inlet or exhaust valves and shall be rated for 100% continuous duty.
- Tip seals shall be of a composite PTFE material and be rated for 8,000 hours operation.
- Compressor bearings shall be external to the air compression chamber and shall all be serviceable for extended compressor life. Bearing maintenance shall not be required until 8,000 run hours. Compressors with bearings that are not accessible for service have a limited life span and shall not be accepted. Compressors shall have an integral radial flow fan for cooling and shall not require any additional electric cooling fans.
- Each compressor shall have flexible connectors on intake and discharge, an electric drive motor, a discharge check valve, an air-cooled aftercooler and a high discharge temperature shut down switch.
- Each enclosure module shall have an isolation valve and a moisture separator with automatic drain.

### Motor

Each compressor shall be belt driven by a 2 pole, TEFC, NEMA construction motor that run at 3500 RPM. Motors are EISA compliant and premium efficient.

### **Motor Slide Base**

Maintenance feature designed for easy adjustment of belt tension from the motor side on the basemount assembly.

- Robust single screw linear belt tension adjustment.
- Custom compact design.

#### Air Receiver

The system shall include an ASME air receiver rated for 165 PSI MAWP or higher. The tank shall be equipped with:

- A pressure gauge and a safety relief valve.
- By-pass valves to allow tank isolation without system shutdown.
- An automatic electronic tank drain with manual override.

The receiver shall be internally lined with an FDA approved material for corrosion resistance.

### **Intake Filters**

The system includes a single inlet filter per compressor stack with a pleated element and a canister. The single inlet filter serves all the compressor modules in the stack. Systems with two compressor stacks will have a second inlet filter serving the other half of the compressors in the system.

The inlet filter system shall be located on the compressor package and plumbed up-stream of the compressor pumps.

## **Desiccant Air Dryers**

Each desiccant dryer shall be sized for the peak calculated system demand to provide a pressure dew point to meet NFPA 99 standards.

- Dryer controls shall include a re-pressurization cycle to prevent shocking of the desiccant bed prior to switching towers.
- An integral purge saving control system shall be provided and shall suspend the purge air loss during periods of low demand.
  When the dryer is in purge control mode, the tower switching valves shall not operate, and only one desiccant bank shall be online. Dryers continue to operate the switching valves in the fixed cycle.

### **Dewpoint Monitor**

Dew point monitor equipped with an LCD dew point display and high dew point alarm with dry contacts for remote monitoring. The dew point sensor (probe) shall be installed so that the monitored airflow is downstream of the pressure regulator assembly. The sensor shall include an auto calibration feature to ensure the accuracy of the dew point measurement without the need to return the sensor to the factory for calibration.



# Medical Scroll Enclosed Air Compressor System with 7.5 & 10 HP Pumps

### **Carbon Monoxide Monitor**

Carbon monoxide (CO) monitor in an enclosure with LCD display of CO concentrations. The monitor shall continuously display the CO content of the discharge air and shall provide audible and visual high CO alarms. High alarm is set at 10 ppm per NFPA99. Dry contacts are provided for remote monitoring of the high CO alarm.

## **Filtration and Pressure Reducing Station**

Each filter/dryer/regulator assembly shall be plumbed with bypass valves to enable service without disrupting air flow to the facility. Each assembly also includes a sample air port. The filtration systems consist of two stages of filtration.

- The first stage of filtration shall include dual pre-filters with element change indicators and automatic condensate drains, installed up-stream of the air dryers.
- The second stage shall include dual particulate filters with element change indicators installed downstream of the air dryers.
- A dual set of pressure reducing valves with pressure gauges shall be installed downstream of the final filters and shall be adjusted to an outlet pressure of 55 psig.

## **Compressor Cabinet Controls**

The controls shall be integrated with the compressor cabinet from the factory. The controls will operate and continuously monitor the system and provide information and alarms to the user through the HMI display. Features include display of system pressure, pump run status, pump fault conditions (high temperature shutdown, motor overload fault), maintenance counters and warnings and system trends. System setup mode allows user to reset alarms and reset maintenance counters. Each compressor pump is automatically staged on or off individually based on actual system demand. Dry contacts are provided for remote monitoring of compressor fault conditions.

Controls to include a building automation communication gateway, through a gateway server card with BacNet® protocol and Web server features. Web server features include email notifications in case the system is in alarm or has achieved one of its maintenance intervals and requires service. An ethernet port is provided for connection to BacNet® server or direct connection to facility Ethernet for viewing of system operations and status via device IP-address.

# NFPA Compliant Master Controller

The system shall include a UL listed master control panel to automatically alternate the lead/lag status of each compressor cabinet and to provide the required NFPA 99 alarms. The control panel shall include a NEMA 12 enclosure with PLC controlled alternation, lighted HOA switch for each compressor cabinet, and reserve compressor in use alarm. A general compressor fault alarm with audible and visual indicators is provided for indication of high temperature shutdown or motor overload fault conditions. All alarms shall have dry contacts on a labeled terminal strip for remote monitoring. All control and alarm functions shall remain energized while any compressor in the system

remains electrically on-line. The lag compressor shall be able to start automatically if the lead compressor fails to operate.

Summary of features:

- NEMA 12 enclosure.
- 115V power circuit required.
- PLC controlled alternation.
- Lighted (LED) HOA switches.
- Alarm horn with acknowledge pushbutton.
- Reserve unit in use light (LED) and dry contacts.
- CO and dew point monitor power and alarm connections.
- · Power connection for tank drain.
- General compressor fault light (LED) to indicate high temperature/ motor overload.
- General fault dry contacts for connection to master alarm.
- Terminal blocks for connection of field devices and remote alarm signals.

## Air-cooled Aftercooler

The compressor system is designed with 3 stages of internal air-cooled aftercooling so to have a maximum approach temperature of 16°F after the tank and 20°F immediately after the compressor above ambient. No additional external aftercooling is required for use with a dryer.

## **Moisture Separator**


A liquid separator with automatic float condensate drain is provided to be installed at the final discharge of each air compressor cabinet and prior to the tank/dryer assembly.


## **Electrical Service Disconnect**


An electrical service disconnect is to be built in for all 460V systems and for 15-30HP 230V systems. Larger 230V systems will require a separate field installed disconnect.





Dimensions											
Model	Dim. A	Dim. A1	Dim. A2	Dim. B	Dim. B1	Dim. B2	Dim. C	Dim C1	Dim. C2	Inlet (x6)	Outlet
MSET50B7	64"	57"	48"	38"	71"	36"	70"	73"	102"	2"	2 ½"
MSET60B7	64"	57"	48"	38"	71"	36"	80"	73"	102"	2"	2 ½"

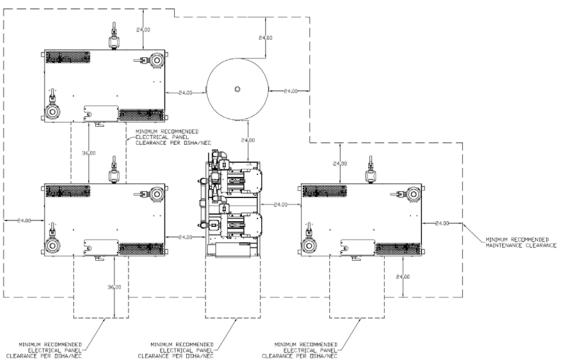


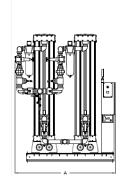


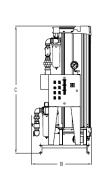

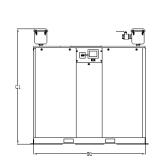

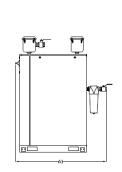

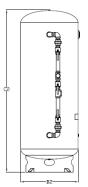

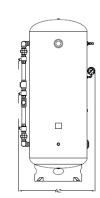

Medical Enclosed Scroll Triplex											
Model	НР	SCFM @	Tank Size (gal)	BTU/Hr <sup>1</sup>	dB(A) Level <sup>1</sup>	System F.L.A. <sup>2</sup>		System Weight			
iviouei		50 PSIG <sup>1</sup>				230V	460V	(lbs)			
MSET50B7	50 (3)	316	400	254,443	68	117	59.5	8,552			
MSET60B7	60 (3)	379.2	400	305,332	72	140	71	9,452			

### Notes:

- 1 –Values are shown with one unit in reserve per NFPA 99.
- 2 System F.L.A. values shown for one enclosure unit; each enclosure gets its own individual power feed.
- 3 3 Year Limited Warranty